

BAPSCARCARE

Professional scar treatment



Frequently Asked Questions

What is a scar?

A scar is a natural reaction of the body to deep damage to the skin, such as after an operation, burn, or accident. A scar is less elastic and lacks some of the functions of healthy skin. In many cases a scar causes discomfort and is unsightly.

When should I use BAPSCARCARE?

- If a scar remains red after the wound closes
- When a fresh scar thickens, grows or causes complaints
- When an older scar causes complaints
- Preventively for closed wounds, when you know that you are sensitive to abnormal scars.

Which BAPSCARCARE product should I use?

That depends on your wound/scar and your personal situation. Consult with your doctor as to which BAPSCARCARE product is most suited to you.

How fast does it work?

You might notice a difference within a few days. However treatment lasts 2 to 6 months on average but therapy can take longer if necessary.

WWW.BAPSCARCARE.COM

Assortment

Silicone scar gel

BAPSCARCARE GEL

Gel
20 g
1 tube



Gel + SPF
10 g
1 tube



Extra thin silicone scar bandage

BAPSCARCARE T

5 x 7 cm
10 pcs



10 x 15 cm
10 pcs



5 x 30 cm
10 pcs



Washable silicone scar bandage

BAPSCARCARE S


Mamma
4 x 30 cm
4 pcs,
2 pair



Mamma
Anchor
4 x 30 cm
4 pcs,
2 pair



Abdomen
4 x 40 cm
2 pcs,
1 pair

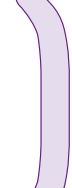

Keyhole
10 x 18 cm
4 pcs,
2 pair



Nipple
Ø 10 cm
4 pcs,
2 pair



5 x 20 cm
2 pcs




10 x 15 cm
2 pcs




15 x 20 cm
2 pcs



40 x 100 cm
1 pcs

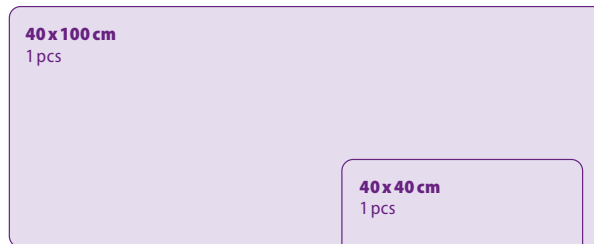

40 x 40 cm
1 pcs

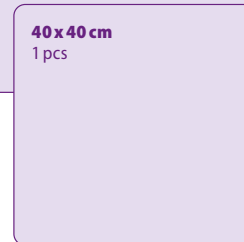

WWW.BAPSCARCARE.COM

BAPMEDICAL
MEDICAL CARE FOR THE SKIN

BAP Medical B.V.
Laan van Westen 64
NL-7336 AZ Apeldoorn

Tel. +31 (0)55 355 25 80
Fax +31 (0)55 355 91 98

www.bap-medical.com
info@bap-medical.com

BAPSCARCARE

Professional scar treatment

*Because
every scar
needs care*

- ✓ Prevents and reduces abnormal scars
- ✓ Reduces complaints (such as itching, redness)
- ✓ Supported by science
- ✓ Comfortably worn

SILICONE THERAPY

Most used by medical specialists!



Professional silicone therapy to prevent scarring and to treat scars

BAPSCARCARE is a medical silicone scar therapy for your skin's faster and prettier recovery. BAPSCARCARE helps existing scars to be flatter, more supple, and thinner. The scar will be less unsightly and in many cases nearly invisible. Even when your scar has turned disfiguring and thick, leaving a "string" across your skin, BAPSCARCARE will reduce it noticeably.

Small and large scars

You can use BAPSCARCARE for small and large scars after:

- Medical operation
- Cosmetic operation
- Burn wounds
- Accidents
- C-Sections
- Abrasions

"I have already tried many silicone gels, but BAPSCARCARE gel is the very best! My scar has largely disappeared and I am bothered less by itching and redness"

- Kim -

Less trouble from your scar

Scars are not only disfiguring but also produce annoying complaints. BAPSCARCARE products relieve and remedy these complaints.

- Itching and dry skin
- Persistent red skin
- Less supple, painful skin
- Thickened skin
- Tightening skin

Supported by science

Because of its proven efficacy, silicone therapy has been the gold standard in scar treatment for decades. Therefore it is the most frequently used therapy by specialists. BAPSCARCARE is a widely recommended therapy as it is so easy to use.

BAPSCARCARE

Professional scar treatment



Which product you need depends on your personal situation and preference. There will be a scar therapy for you!

Silicone scar gel with UV protection

Scars are sensitive to UV radiation, especially when scars are easily exposed to the sun like the neck and face area. Scars exposed to UV rays suffer an increased risk of permanent discolouration and skin cancer for a period of at least a year after the wound has closed. BAPSCARCARE GEL with high UV protection (SPF 40) provides optimal treatment and protection - especially when you cannot avoid exposure to the sun.



"With BAPSCARCARE T I have to apply a new bandage every 7 days. It is also much thinner. Super!"

- Merel -

"BAPSCARCARE is user-friendly and very comfortable!"
- panel study by specialists and
randomly selected trial subjects -

High ease of use

We have given as much attention to the ease of use of BAPSCARCARE, as to its effect. That has resulted in the unique user-friendly characteristics of our products. Here are all the benefits:

BAPSCARCARE GEL

Silicone scar gel

- Feels pleasant and silky smooth
- Invisible - ideal for use on neck and face
- Dries immediately
- Not sticky
- Can be used with make up
- Also available with UV protection



BAPSCARCARE T

Extra thin silicone scar bandage

- Washing not necessary
- Extra thin and virtually invisible
- Self-adhering
- Ideal for skinfolds and joints



BAPSCARCARE S

Washable silicone scar bandage

- Easily washed / quick drying
- Transparent
- Self-adhering
- Comfortable under pressure garment therapy
- Available in pre-shaped designs (see product line)

